



# DURACETAL<sup>TM</sup>

COSMETIC APPLIANCES

## DURACETAL<sup>®</sup> PATIENT CARE

With proper care, your appliance should provide years of use. Please follow these simple recommendations:

- Place dental appliance in water or denture cleanser when not being worn to help it retain shape, remain pliable, and prevent it from drying out.
- Do not place in hot water as it may cause warping.
- Do not attempt to chew ice or meat bones while wearing appliance.
- Clean dental appliance daily.
- Take care not to drop appliance when cleaning.
- Do not use denture cleansers while appliance is in the mouth.
- Do not clean using a stiff brush or abrasive toothpaste.
- Do not clean appliance with bleach or powdered household cleansers as they can be abrasive and might damage the appliance.
- Commercial denture cleansers for removeable appliances come as tablets, creams, pastes, gels, and solutions.  
Follow the manufacturer's instructions after rinsing away loose food particles and removing any denture adhesive.
- Staining of dental appliances may occur from some foods. Over-the-counter denture cleansers can remove these superficial stains. It is safe to use a cleanser for more than the recommended time to remove stubborn stains.
- Appliances can also be cleaned with gentle toothpaste, mild hand soap or dishwashing liquid using warm water and a soft-bristle toothbrush.



**Classic★Craft**  
DENTAL LABORATORY

877.552.7238 Fax: 912.355.1007  
classiccraftdental.com

©2022 CLASSIC CRAFT DENTAL LABORATORY

